

The description of the product

Visor Pro Elite 2 00905267 ja 2S 009052676

The disposable Visor Pro Elite 2 / 2S is intended for to prevent liquid droplets and splashes from the front entry to face. The visor covers chin, mouth, nose and eyes. The visor is intended for to be used for example by health care professionals, store personnel, food delivery and manufacturing, home health care and lay user. The visor can be used in combination with eyeglasses.

The visor is in conformity with the relevant Union harmonization legislation Regulation (EU) 2016/425.

The visor meets the requirements of standard EN 166:2001 (excluding clause 7.1.5.2) Personal eye-protection, Specifications.

The notified body SATRA Technology Europe Ltd nro 2777, in Bracetown Business Park, Clonee, Dublin 15, Dublin, performed the EU type-examination (Module B) and issued the EU type-examination certificate (2777/20191-01/E00-00).

The visor's EU Declaration of Conformity is available at: www.screentec.com/quality

Manual

1. Take the visor out of the plastic bag
2. Stretch the rubber band with your hand
3. Put visor to the face, cushion against forehead
4. Put the rubber band to the back of the head
5. Set the visor to cover chin, mouth, nose and eyes



1.



2.



3., 4. and 5.

Note:

- The visor can be worn with eyeglasses or goggles
- The visor is disposable
- To optimise protection, it will be necessary to use these product with suitable gloves/gowns/masks. In this case, before carrying out the risk-related activity, consult your supplier or supervisor to ensure that all your protective products are compatible and suitable for your application.
- The visor should be disposed of and replaced if badly scratched or damaged in some way that prevents safe use.

Warnings:

- The visor does NOT tightly cover the face.
- The visor does NOT prevent liquid droplets and splashes from the side NOR below.
- The visor does NOT provide protection against aerosols.
- The visor does NOT protect against biological threats.
- The visor does NOT withstand strong acids, alkali NOR alcohols.
- The visor is NOT impact or strike resistant.
- The visor does NOT withstand hot droplets and splashes of liquid.
- The visor does NOT protect against open flames or sparks
- Always remember that no item of PPE can provide full protection and care must be always be taken while carrying out the risk-related activity. Never use this visor for protection against other hazards for which the product is not design for or tested against.
- Parts of the visor that come into contact with your skin may lead to allergic reactions. In this case please discontinue use and seek medical advice.

Storage:

5 pieces of the product is packed in a bag that protect them from dust.

The product is recommended to be stored at normal room temperature and humidity (+15 - +25 ° C, RH < 50 %), protected from light.

Note! Shelf life of the product 3 years from the date of manufacture (time of manufacture in the transport packaging of the product wwww).

Maintenance:

Visor Pro Elite 2 00905267 ja 2S 009052676 is disposable, no maintenance or cleaning recommended.

Disposal:





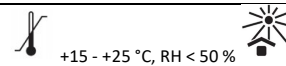
Visor Pro Elite 2 00905267 ja 2S 009052676 is disposed within the energy waste.

Technical information:

Size	One size
Material of the visor rubber band	Pre-shrunk woven rubber band with natural rubber thread, PES, rubber (OBS. NOT latex-free)
Width of the visor rubber band	30 mm
Material of the forehead cushion	Foam rubber. No allergenics, no volatile compounds. The material does not release dust particles.
Thickness of the forehead cushion	41 mm
Lenght of the forehead cushion	370 mm
Color of the forehead cushion	Varies
Material of the visor sheet	Clear PET
Width of the visor sheet	410 mm
Length of the visor sheet	Visor Pro Elite 2 00905267 280 mm Visor Pro Elite 2S 009052676 230 mm
Thickness of the visor sheet	0,325 mm



Markings on the visor and in the product box and explanation of the markings:

Manufacturer's logo:		
Contact info:	www.screentec.com info@screentec.com	
Product's name:	Visor Pro Elite 2 00905267 tai Visor Pro Elite 2S 009052676	
Ocular marking:	Scr 1 S - EN 166 3 S	where:
	Identification of the manufacturer:	Scr
	Number of standard:	EN 166
	Optical class:	1
	Field use:	3 (liquids)
	Symbol of increased strength:	S
CE-marking and applicable standard:	 CAT II EN166:2001	
The product is disposable:		
Read the operating instructions		
Storage	 +15 - +25 °C, RH < 50 %	

For further questions, please contact the manufacturer:

Screentec Oy
 Konekuja 2
 90620 Oulu
 FINLAND
 p. +358 8 561 2000
 info@screentec.com

